





Before You Start

1 **60 minutes prior to the test do not brush your teeth, eat, drink, smoke, use mouthwash, chew gum, or floss.** Any of these could result in an invalid test. You are now ready to begin. 

2 If you are on-site, sanitize your hands. If at home, wash your hands thoroughly for 20 seconds with soap and dry them. 

Provide Saliva Sample - Video Instructions can be found <https://youtu.be/QdHZaoP5pEk>

3 Unscrew the cap from the vial and set it aside. 


4 Retrieve the funnel from the kit and attach it securely to the top of the vial. 

5 Drool 1.0 ml to 1.5 ml of saliva (excluding bubbles) into the vial. Take your time. Do NOT spit. Most individuals take 2-5 minutes to produce sufficient clear saliva. If you have difficulty try one of these tips:


- Gently rubbing the outside of your cheeks
- Making chewing motions with your mouth
- Smelling or imagining sour foods such as lemons
- Thinking about your favorite foods


Important: For proper results, saliva must be colorless and free of particles.

Take your time.

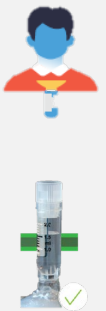



Children under 16 should have adult assistance during collection.


6 Remove the funnel and discard it. Do not return it with your sample. 


7 Screw the cap back onto the vial. Firmly tighten to ensure cap and vial is completely sealed. Wipe the vial with the provided alcohol wipe. 

Return Kit for Testing

8 If you are on-site, sanitize your hands. If you are at home, wash your hands thoroughly for 20 seconds with soap and dry them. 

9 If required, write your birth month and day (MM/DD) or full birthday (MM/DD/YYYY) on the vial with an industrial-grade marker such as a Sharpie. 


10 Place the absorbent material and collection vial into the biohazard bag and seal tightly. 

11 Protect the bag from temperature extremes and return on the same day it was collected to your local collection receptacle as directed. 

Warning: Keep away from temperature extremes.

Your sample must be returned within 12 hours of creating it.

Do NOT freeze or keep in your car.



This product has not been FDA cleared or approved but has been authorized for emergency use by FDA under an EUA for use by authorized laboratories. This product has been authorized only for the detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens. The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal, Food, Drug, and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated, or authorization is revoked sooner.