

Thank you for choosing covidSHIELD for your Covid-19 test. Your test kit contains the following items:




- Protect from temperature extremes: do not store kits in your car.
- Your sample must be returned within 12 hours of creating it.

For assistance, please call customer support at 833-762-0762.

1 Get Ready

Read the Instructions for Use (IFU) sheet in your test kit carefully. Then **SCAN** the QR code to view further instructions on how to provide a good sample. Please rewind and rewatch the video as needed so you understand the process fully.

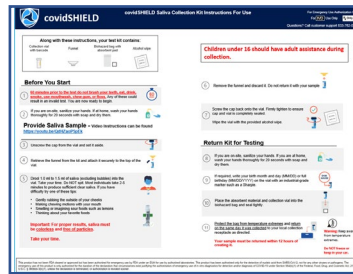

2 Activate Your Kit

To Activate your test kit with an existing account, log onto the Shield T3 Covid registration site using the link provided in an email from your organization.

To setup a new account, use the link and agency code, provided in an email from your organization.

Once logged onto your existing account or created new account, you will enter the bar code number and the number below the barcode located on vial in the kit.

Congratulations, your kit is now activated!

3 Provide Your Sample

Provide your sample by drooling into the vial as instructed. When you are finished, screw the cap back onto the vial, sanitize the vial and your hands with the alcohol wipe, and place the absorbent pad and sealed vial in the small biohazard bag. Seal the bag and discard all other materials.

4 Return Your Sample

Return your sealed small biohazard bag to a designated collection point as instructed. Drop your biohazard bag into the collection device; do not touch any other biohazard bags.

Remember: You must return your sample within 12 hours of creating it, and the sample must be protected from temperature extremes.